


Xpert® Xpress MVP Cartridge Preparation

Refer to the Instructions for Use for detailed instructions, precautions, and warnings.

- 1** Take one cartridge and transfer pipette (provided).

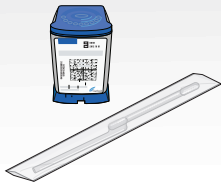


- 2** Vigorously shake the transport tube 3 to 4 times.



- 3** Open the cartridge lid.



- 4** Open the lid of the specimen transport tube. Squeeze the bulb of the transfer pipette completely. Holding it fully flat, place the pipette tip in the specimen transport tube.

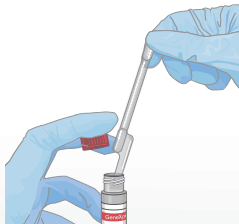

- 5** Keeping the pipette below the surface of the liquid, release the bulb slowly to fill the pipette before removing from the tube.



- 6** Squeeze the bulb of the pipette completely to empty the contents into the cartridge. Close the cartridge lid.

